Title 8 - Initial Fit-test

- Explanation from CAHF DPP
- Grant funded program
- Tools, templates, resources
- www.cahfdisasterprep.com
- Jason Belden, Director of Emergency Preparedness and Physical Plant Services
- jbelden@cahf.org



- <u>California Code of Regulations, Title 8, Section 5144. Appendix A Fit Testing</u> <u>Procedures (Mandatory).</u>
- Employer shall conduct fit testing using the following procedures
- 1. Employee given choice of respirator from available supply limited supply considerations
- 2. Employee is given demonstration of how to don/doff and adjust respirator -Demonstration of Donning (Putting On) Personal Protective Equipment (PPE) - YouTube
- 3. Let employee know that if the respirator fits it will confer adequate protection
- 4. Have employee choose respirator that works best for them



5. Have employee spend five minutes assessing comfort; have them don and doff multiple times to become adept at adjusting tension on straps

6. Assess position of mask (nose, cheeks, face); room for eye protection; room to talk

7. Have employee self-observe in a mirror – Chin placement, strap tension, fit across the nose bridge, proper size

8. Seal Check - Employee don's mask; turns head side-to-side and up and down while taking deep breaths



9. Limited facial hair allowance - <u>Infographic - Facial Hairstyles and Filtering</u> <u>Facepiece Respirators (cdc.gov)</u>

10. Refer to MD if difficulty breathing while wearing mask to determine capability <u>CAHF Model Respiratory Protection Program</u>

- 11. If respirator does not fit employee given another choice of mask to retest
- 12. Describe testing procedures to employee
- 13. Employee wears all PPE they will be required to wear with respirator



14. Test exercises –

- a. Breath normally
- b. Breath deeply
- c. Move head side-to-side up, up-and-down; range of motion
- d. Talking repeat passage
- e. Employee smiles, grimaces, frowns with mask on
- f. Employee jogs in place or touches toes
- g. Employee finishes by breathing normally after test with respirator on

Qualitative process - <u>3M Respirator Fit Kit Test Video - YouTube</u>

